**REF** PM-962 Aluminum Orthopedic Hand, Large, Contains 1  
PM-963 Aluminum Orthopedic Hand, Extra Large, Contains 1  
Made in USA

**STERILIZE BEFORE USE**

**DISCARD AND REPLACE DEVICE IF COATING IS CRACKED**

## STERILIZATION INSTRUCTIONS

Device must be cleaned in accordance with standard institutional protocol prior to sterilization.

Cycle Type: Pre-Vac Steam Sterilization Full Cycle Instructions

Double wrapped in standard central supply wrap

Sterilization Temperature: 132°C

Sterilization Exposure Time: 4.0 Minutes

Cycle Type: Gravity Steam Sterilization Full Cycle Instructions

Double wrapped in standard central supply wrap

Sterilization Temperature: 121°C

Sterilization Exposure Time: 30 Minutes

## INSTRUCTIONS

Place patient's hand on top of the Aluminum Hand.

Bend wrist tabs over patient's wrist and bend finger tips over patient's fingers as needed. The patient's hand should be flat and immobilized.

After procedure, straighten Aluminum Hand fingers and wrist tabs.

Aluminum Hand must be cleaned in accordance with standard institutional protocol prior to sterilization.

Sterilize according to "Sterilization Instructions" provided.

Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

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Miltex is a registered trademark and Miltex Redefining Excellence is a trademark of Miltex, Inc., a subsidiary of Integra LifeSciences Corporation. The Integra wave logo is a trademark of Integra Life Sciences Corporation.
Copyright © 2008 Integra LifeSciences Corporation