Rx ONLY

Rx ONLY

Rx ONLY


REF 75-12

Precautions

- This is a stationary head type instrument; users should not remove the head from the sheath.
- Do not put angle through ultrasonic disinfection solution cleaning procedure.
- Do not use cold sterilization or dry heat.
- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.

Lubrication Instructions

1. After thoroughly cleaning the outside of the angle with a disinfectant moistened gauze sponge, open the latch and insert two drops of oil into hole "A" and hole "B".

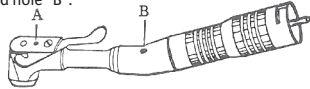2. Close the latch and rotate the shaft several times to circulate the oil. Wipe off any excess oil expelled from the oil holes.
3. Sterilize by autoclaving before putting back into service.

Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250°F (121°C) with a 30 minute exposure time.

Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

0297

Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA  
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax  
integralife.com/integra-miltex

Miltex GmbH • Mühlstr. 6 • 78604 Rietheim-Weilheim • Germany  
Phone: 49 7461 /9634-14 • Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.

© 2011 Integra LifeSciences Corporation. All rights reserved.

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