Integra™ Miltex®
Mid Grade

Autoclavable Prophy Angles


Directions for Use

Rx ONLY

REF 75-28 Screw Type REF 75-30 Snap On

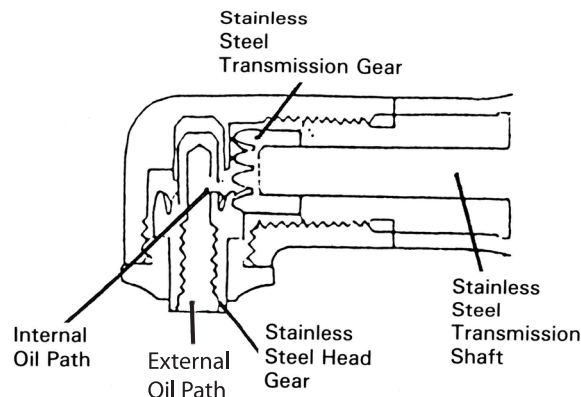Description

- Fully autoclavable. Recommended temperature is 250°-270°F, 15-27 psi for 20 minutes.
- Both screw type and snap type can be oiled without disassembling the head.
- The gears and the shaft are made of hardened stainless steel.
- Phosphor bronze cup and head bearings.

Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.

Lubrication Instructions

- Remove cup or brush.
- Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
- Insert three drops of oil into the appropriate hole:

- Screw Type** - The hole where the cup or brush is screwed in.

- Snap Type** - The hole in the middle of the knob onto which the cup is attached.

- After oiling, rotate the shaft to circulate the oil within the head.
- Sterilize by autoclaving before putting back into service.

Cleaning/Sterilization Instructions

- Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
- Grasp the knurled part of the head cap with the wrench provided and turn wrench clockwise. Remove the stainless steel head gear. Examine the oil hole to make sure it is not clogged. Clean all parts. Reassemble by first inserting the head gear. Attach cap and turn cap counterclockwise with your fingers. To make sure that the gears are intermeshed properly, turn the transmission shaft with your fingers checking for smooth rotation. Using wrench provided, tighten cap turning counterclockwise.
- Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250° F (121° C) with a 30 minute exposure time.

Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

Rx ONLY

Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax
integralife.com/integra-miltex

EC REP
Miltex GmbH • Mühlstr. 6 • 78604 Rietheim-Weilheim • Germany
Phone: 49 7461 /9634-14 • Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.
© 2011 Integra LifeSciences Corporation. All rights reserved.

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