
Autoclavable
Ball-Bearing Spring Latch,
Cartridge 40,000 rpm, Contra Angle
Directions for Use

Rx ONLY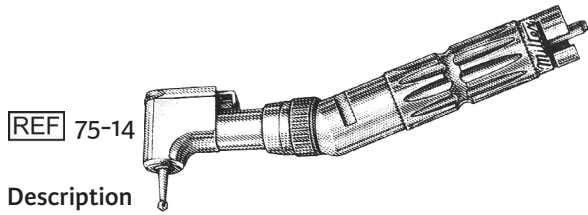

REF 75-14


Description

- Two precision ball bearings are fitted into a cartridge in the head for smooth rotation of the bur and minimal generation of heat.
- Easy replacement of worn-out ball bearings.
- Bur holding mechanism enables loading and unloading of burs with just a push of the thumb.
- Sectional head can be fixed in any angle.


Precautions

- Do not put angle through ultrasonic disinfection solution cleaning procedure. Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.
- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.

Component Parts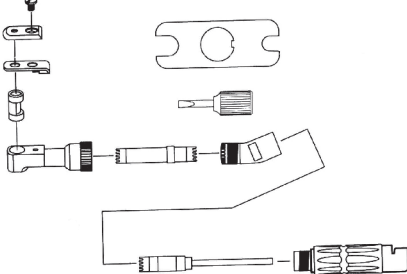

Lubrication Instructions

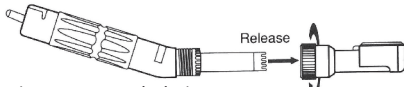1. Turn union nut counterclockwise and remove contra angle head from sheath.

2. Using screwdriver, remove screw so the latch cover can be taken off.

3. Take off latch pulling it toward sheath. There is a spring rest and spring under the head.

4. Using screwdriver push out cartridge inserting the driver into bore of gear.

5. After thoroughly cleaning the outside of the angle with a disinfectant moistened gauze sponge, immerse the angle in the oil provided and run at low speed for 1 minute.

6. Sterilize by autoclaving before putting back into service.

Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250° F (121° C) with a 30 minute exposure time.

Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

Rx ONLY

Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA ■ 717-840-2763 outside USA ■ 717-840-9347 fax
integralife.com/integra-miltex

Miltex GmbH ■ Mühlstr. 6 ■ 78604 Rietheim-Weilheim ■ Germany
Phone: 49 7461 /9634-14 ■ Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.
© 2011 Integra LifeSciences Corporation. All rights reserved.

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