THE OS LOCATOR

One molded band above the tactile handle and opposite the distal end of the device identifies the disposable Os Locator. The Os Locator may be used to find and gently dilate the external os. Gradual dilation in combination with its tip flexibility and smooth surface will ensure maximum patient comfort. The Os Locator is tapered 23 mm to an intermediate diameter of 3.8 mm.

THE CANAL DILATOR

Two molded bands above the tactile handle identify the Canal Dilator from the Os Locator and the Fundus Sound. Its longer tapered tip is 45 mm and extends to an intermediate diameter of 3.8 mm. The Canal Dilator may be used to slowly dilate the length of the cervical canal. Its flexible and smooth surface will provide maximum patient comfort.

THE FUNDUS SOUND

Three molded bands opposite the distal tip identify the Fundus Sound from the others as described above. The tapered length from the distal tip is 85 mm. The smooth and flexible Fundus Sound is designed to pass through the entire length of the cervix, dilate the internal os and sound the fundus to determine uterine depth.

PRODUCT NUMBERS

REF	30-5001 Disposable Os Locator, Sterile
	30-5002 Disposable Canal Dilator, Sterile
	30-5003 Disposable Fundus Sound, Sterile

MILTEX® CERVICAL DILATORS (Latex-free)

Miltex® Disposable Cervical Dilators are designed for single-patient use only. They are made of medical-grade polyethylene.

CAUTION

R ONLY Federal (US) law restricts this device to sale by or on the order of a physician.

INDICATIONS

These Cervical Dilators are especially useful in locating and dilating the external os, the cervical canal and the internal os. They are particularly valuable instruments to be used on post-menopausal patients with cervical stenosis.

CONTRAINDICATIONS

Cervical dilators should not be used under the following conditions:

- Acute genital track or pelvic infections.
- A pregnancy or the suspicion of a pregnancy.
- Any invasive cancer that is visible upon examination.
- Any non-compliant patient.

Miltex® Disposable Cervical Dilators are smooth and flexible, light in weight and can be easily inserted and removed for their intended applications. Please be aware that excessive force should be avoided and these instruments should not be used for prolonged periods of time.

Each dilator is manufactured to exacting standards. They are flexible, very smooth and deliver excellent patient comfort for their intended indication.

PRECAUTION

Check the packaging for damage for this may compromise sterility. Visually inspect the dilators for any surface damage.

SYMBOLS USED ON LABELING

Caution

Product complies with requirements of directive 93/42/EEC for medical devices

Consult Instructions for Use

Manufacturer

Use by: Year-Month

Authorized Representative in the European Community

Batch Code / Lot Number

Catalog Number

Single Use Only Do Not Reuse

Latex-free

STERILE – STERILIZED BY IRRADIATION Sterility guaranteed unless package opened or damaged. Do not re-sterilize.

Rx Only: Caution: Federal (US) law restricts this device to sale by or on the order of a physician.

Manufactured for:
Miltex, Inc.
589 Davies Drive
York, PA 17402 USA
toll free phone: 866-854-8300
phone: 717-840-9335
fax: 717-840-9347
www.miltex.com

Manufacturer:
Panpac Medical Corporation
6F-2, No. 202, Sec. 3, Ta-Tong Rd
Shi-Chih City, Taipei Hsien
Taiwan, R.O.C.
Phone: (886) 28647-1300
Fax: (886) 28647-2770

Nunzio Fracalanza
Via M. Molini 12, 35023
Bagnoli Di Sopra, Padova
Italy
Phone: +39 346 06 23 641

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