# Integra™ Miltex® Deluxe Autoclavable “FG” Contra Angle Directions for Use

# INTEGRA™

**Rx ONLY**

**REF** 75-16

## Description


- Fully autoclavable.
- The gears and the shaft are made of hardened stainless steel.
- Phosphor bronze cup and head bearings.

## Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.

## Lubrication Instructions


1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Remove head cap and gear.
3. Place three drops of oil on the head gear and in oil hole in body as indicated.
4. Reassemble unit. Rotate the shaft several times to circulate the oil. Wipe off any excess oil expelled from the oil hole.
5. Sterilize by autoclaving before putting back into service.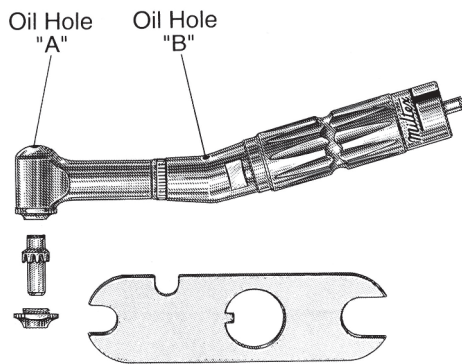

## FG Contra Angle Bur Insertion and Removal


### 1. Insertion of Bur

- Using black cushioned insertion/removal tool, push bur until it seats completely.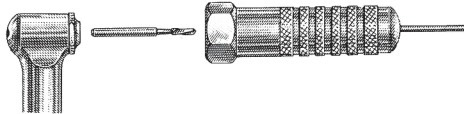

### 2. Removal of Bur

- Using pin end of special bur insertion/removal tool, push bur out of chuck by inserting pin end into hole in back of the head and push until bur is expelled.

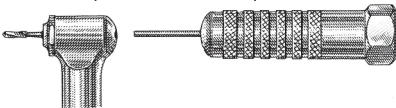## Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250° F (121° C) with a 30 minute exposure time.

## Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

**Rx ONLY**

Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA  
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax  
integralife.com/integra-miltex

**EC REP**  
Miltex GmbH • Mühlstr. 6 • 78604 Riethem-Weilheim • Germany  
Phone: 49 7461 /9634-14 • Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.  
© 2011 Integra LifeSciences Corporation. All rights reserved.

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Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402 USA
Miltex GmbH ■ Mühlstr. 6 ■ 78604 Rietheim-Weilheim ■ Germany