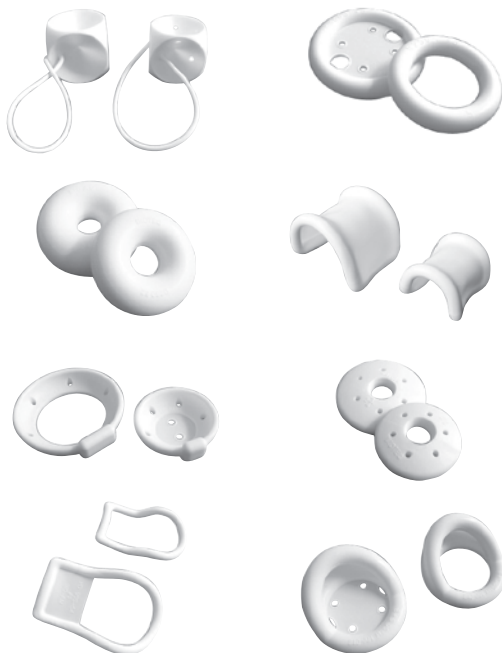
PESSARY

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Instructions for Use/English

INDICATIONS FOR USE

The flexible silicone pessary is commonly used for a mild first or second-degree uterine prolapse or proidentia. **Pessaries are designed for single patient use and supplied non-sterile.**

CAUTION

R_{ONLY} Federal (US) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

Pessaries are contraindicated in acute genital tract infections, pelvic infections, or non-compliant patients.

PROCEDURES

Fitting usually requires a trial of various sizes to determine the proper pessary size. Pessary Fitting Set is a valuable aid in selecting the correct pessary. A vaginal lubricant may be required for the patients comfort before insertion of a pessary. The physician may suggest K-Y® jelly, Replens® or Premarin®. (Premarin® requires a prescription).

The physician should determine if the patient requires estrogen therapy before prescribing a pessary.

1. Perform a normal pelvic examination prior to the fitting and introduction of a pessary. A pelvic exam helps determine the appropriate size. The pessary will easily fold across the largest drainage holes to simplify insertion and removal. The pessary should be large enough for its indication, yet not cause any undue pressure or discomfort.
2. Compress the pessary and gently insert the pessary through the introitus. Once the pessary has passed the introitus, release the compressed pessary so that it expands to its normal shape.
3. Gently move the pessary to the cervix to support the cystocele or rectocele and the uterus. When in the correct position, the physician should be able to insert an examining finger between the outer edge of the pessary and the vaginal wall. This spacing will insure the patients comfort and reduce the risk of pressure necrosis.
4. Once in place, the pessary should not dislodge when standing, sitting, squatting or bearing down. It should not be uncomfortable for the patient during any of these routine activities.
5. When fitting a pessary, be sure to instruct the patient on its cleaning, insertion and removal. The patient should perform these tasks before the first follow-up visit.
6. Tell the patient to report any discomfort immediately.

REMOVAL, CLEANING AND SUGGESTED FOLLOW-UP

1. To remove the pessary, carefully move the pessary forward toward the introitus. When the pessary is within reach, compress the pessary and carefully withdraw.
2. The pessary should be removed for cleaning every day or two. The pessary can be cleaned with mild soap and warm water.
3. Advise the patient on pessary usage during sexual activity.
4. Discuss the importance of following instructions and the expected length of time for pessary use.
5. Within 24 to 48 hours, be sure the patient is not allergic to the pessary. Examine the vagina and ask the patient if there has been any discomfort, irritation, pressure, sensitivity, or unusual vaginal discharge. Also determine if there has been any improvement in her personal symptoms.
6. Schedule follow-up visits to fit the needs of the patient.

Miltex is a registered trademark and Miltex Redefining Excellence is a trademark of Miltex, Inc., a subsidiary of Integra LifeSciences Corporation. The Integra wave logo is a trademark of Integra Life Sciences Corporation. K-Y is a registered trademark of Johnson & Johnson Corporation. Replens is a registered trademark of L'il Drug Store Products, Inc. Premarin is a registered trademark of Wyeth Corporation. Copyright © 2008 Integra LifeSciences Corporation

Manufactured for:

Miltex, Inc.
589 Davies Drive
York, PA 17402 USA
toll free phone: 866-854-8300
phone: 717-840-9335
fax: 717-840-9347
www.miltex.com

Manufacturer:
Panpac Medical Corporation
6F-2, No. 202, Sec. 3, Ta-Tong Rd
Shi-Chih City, Taipei Hsien
Taiwan, R.O.C.
Phone: (886) 28647-1300
Fax: (886) 28647-2770

Nunzio Fracalanza
Via M. Molini 12, 35023
Bagnoli Di Sopra, Padova Italy
Phone: +39 346 06 23 641

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