INSTRUCTIONS FOR USE

Rx Only CAUTION: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

PRECAUTIONS

- Instruments/devices should be inspected for proper function prior to use.
- For GripLite® Instruments and hand files, do not use dry heat sterilization or expose to phenols or iodophors

CLEANING INSTRUCTIONS

Automated cleaning

1. Place instrument/device into the ultrasonic basket, sterilization tray or cassette.
2. Place the ultrasonic basket of instruments/devices, sterilization tray or cassette into the ultrasonic cleaner or washer.
3. Follow the manufacturer's instructions for the appropriate cleaning cycle for the instruments/devices being cleaned.
4. Thoroughly rinse the instrument/device, sterilization tray or cassette under warm tap water.
5. Inspect the instrument/device for any remaining debris and repeat steps 1 through 4 if debris remains on the instrument/device.
6. Allow the instrument/device to completely dry prior to proceeding to the sterilization process.

Manual cleaning

1. Remove any visible debris or foreign matter from the surface of the instrument/device by scrubbing the instrument/device with a cleaning brush and a pH 7 detergent such as Integra™ Miltex® Surgical Instrument Cleaner (3-720, 3-725 or 3-726).
2. Thoroughly rinse the instrument/device under warm tap water.
3. Inspect the instrument/device for any remaining debris and repeat steps 1 and 2 if debris remains on the instrument/device.
4. Allow the instrument/device to completely dry prior to moving to the sterilization process.

STERILIZATION INSTRUCTIONS

- Sterilize files according to an approved National/International Standard. Integra™ Miltex® files have been tested with no adverse effects under the following sterilization parameters in a standard table top style sterilizer:

Temperature: 250°F (121°C)

Exposure Time: 30 minutes

For best results: Do not sterilize size 140 files with stops in place.

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