PRODUCT NUMBERS

REF	30-3001 Silicone Vaginal Dilator Set Large
	30-3002 Silicone Vaginal Dilator Set Medium
	30-3003 Silicone Vaginal Dilator Set Small

INDICATIONS

The dilators may be used to enlarge the vagina for an examination, a surgical procedure, or for the relief of vaginismus. Vaginal Dilators are designed for single patient use and are supplied non-sterile.

CONTRAINDICATIONS

Dilators are contraindicated in acute genital tract infections, pelvic infections, or non-compliant patients.

CAUTION

R Federal (US) law restricts this device to sale by or on the order of a physician.
ONLY

PROCEDURES

For the indications, it is usually desirable to either support or distend the vaginal canal. Dilators are devices that can be used for this purpose. They have a smooth surface, are flexible, light in weight, shaped to stay in place, and can be easily inserted and removed by the patient.

A vaginal lubricant may be required for the patient's comfort before insertion of a dilator. The physician may suggest K-Y® jelly, Replens® or Premarin®. (Premarin® requires a prescription.)

INSTRUCTIONS

1. Perform a normal pelvic examination prior to the fitting and introduction of a dilator. A pelvic exam helps determine the appropriate size.
2. The length of time a patient should use a vaginal dilator is usually from 5 to 20 minutes per day. The physician may alter the frequency and duration of dilator usage as the prescribed therapy proceeds to the larger sizes in the set.
3. Once the dilator family size or set is determined, the smallest dilator should be inserted into the vagina with the base or flange in the downward position.
4. The dilator should be large enough for its indication, yet not cause any undue pressure of discomfort.
5. The patient should be instructed to gradually increase the dilator size over a suggested period of time until the patient is comfortable with the insertion of the largest size in the set.
6. Tell the patient to report any discomfort immediately.

REMOVAL, CLEANING, AND SUGGESTED FOLLOW-UP

1. To remove a dilator, gently pull downward on the exposed flange until the dilator is completely removed.
2. After removal, the dilator can be cleaned with mild soap and warm water. (The physician may follow the suggested disinfecting procedures.)
3. Discuss the importance of following instructions and the expected length of time for dilator use.
4. Within 24 to 48 hours, be sure the patient is not allergic to the dilators. Examine the vagina and ask the patient if there has been any discomfort, irritation, pressure, sensitivity, or unusual vaginal discharge. Also determine if there has been any improvement in her personal symptoms.
5. Schedule follow-up visits to fit the needs of the patient.

INSTRUCTIONS FOR THE PATIENT

Your physician has recognized the need for you to use a set of silicone vaginal dilators. As instructed by your physician, begin your vaginal therapy with the smallest dilator. When it is inserted properly into the vagina, you may be conscious of a mild feeling of pressure. Gradually, over a period of time, your physician will advise you to select the next dilator size for insertion and so on until you are completely comfortable using the largest dilator in the set. At any time, should the dilators become uncomfortable, promptly notify your physician. Follow "Removal, Cleaning and Suggested Follow-up" Steps 1-2.

PHYSICIAN SUGGESTED DISINFECTING PROCEDURE

1. Autoclave at 15 lbs. of pressure at 250°F or 121°C for a period of 10 minutes.
2. Boil for 15 minutes.
3. Cold sterilize with an approved agent, e.g. Cidex®, and then thoroughly rinse with water.

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Miltex is a registered trademark and Miltex Redefining Excellence is a trademark of Miltex, Inc., a subsidiary of Integra Life Sciences Corporation. The Integra wave logo is a trademark of Integra Life Sciences Corporation. K-Y and Cidex are registered trademarks of Johnson & Johnson Corporation. Replens is a registered trademark of L'il' Drug Store Products, Inc. Premarin is a registered trademark of Wyeth Corporation. Copyright © 2008 Integra Life Sciences Corporation

Manufactured for:
Miltex, Inc.
589 Davies Drive
York, PA 17402 USA
toll free phone: 866-854-8300
phone: 717-840-9335
fax: 717-840-9347
www.miltex.com

Manufacturer:
Panpac Medical Corporation
6F-2, No. 202, Sec. 3, Ta-Tong Rd
Shi-Chih City, Taipei Hsien
Taiwan, R.O.C.
Phone: (886) 28647-1300
Fax: (886) 28647-2770

Nunzio Fracalanza
Via M. Molini 12, 35023
Bagnoli Di Sopra, Padova Italy
Phone: +39 346 06 23 641

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