Integra™ Miltex® Deluxe Autoclavable

“U” Contra Angle - 20,000 rpm

Directions for Use

Rx ONLY

CE
0297

REF 75-15

Description


- Fully autoclavable.
- The Contra Angle can be oiled without disassembling.
- The gears and the shaft are made of hardened stainless steel.
- The special latch prevents oil from escaping during use.
- Phosphor bronze cup and head bearings.

Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.

Lubrication Instructions

1. After thoroughly cleaning the outside of the angle with a disinfectant moistened gauze sponge, open the latch and insert three drops of oil in hole “A” and hole “B”.

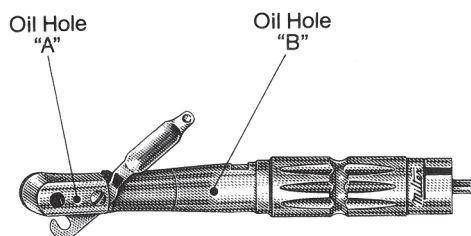2. Close the latch and rotate the shaft several times to circulate the oil. Wipe off any excess oil expelled from the oil hole.
3. Sterilize by autoclaving before putting back into service.

Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. The contra angles have been tested with no adverse effects at 250°F (121°C), with a 30 minutes exposure time.

Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

0297

Rx ONLY

Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA ■ 717-840-2763 outside USA ■ 717-840-9347 fax
integralife.com/integra-miltex

Miltex GmbH ■ Mühlstr. 6 ■ 78604 Rietheim-Weilheim ■ Germany
Phone: 49 7461 /9634-14 ■ Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.
© 2011 Integra LifeSciences Corporation. All rights reserved.

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Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402 VS
Miltex GmbH ■ Mühlstr. 6 ■ 78604 Rietheim-Weilheim ■ Duitsland
© 2011 Integra LifeSciences Corporation. Alle rechten voorbehouden.